

# GREEN PRESS 12

- A device for medical and cosmetic lymph drainage with 2 x 12 exits
- The ideal tool for the therapy of various lymphatic edemas
- Highly effective basic anti-cellulite therapy
- Prevention of swollen feet and cramp veins 7 standard and 14 free programs
- Adjustable drainage parameters – pressure, compression and pause duration, therapy time

CE 2265



# GREEN PRESS 12

The GREEN PRESS 12 has been designed on a principle of manual lymph drainage. The special cuffs are controlled by microprocessor what allows a selection of the precise treatment parameters. There are seven pre-set programs for optimal therapy of different conditions. Additional 14 programs are free to be created by the user according to her/his experience and needs. The cuffs consist of compartments which partially overlap what enables lymph drainage in the most natural way. The compartment pressure is adjustable and can be easily controlled via front board of the instrument.

Two cuff models are available with the machine. They both allow treatment of the arms (one or both simultaneously), legs, buttocks and abdomen, the only difference is in design and material they are made of.

The Green Press 12 is an excellent additional therapy to the manual lymph drainage in every physiotherapy unit. Its therapeutic effect is a valuable contribution to the treatment of early as well as advanced stages of various pathologies. When supervised by an expert, it can be also used in cosmetic salons for effective treatment of all cellulite stages. In a combination with other established cosmetic procedures (ultrasound therapy, electrotherapy, anti-cellulite creams, etc.) the final result in reducing the cellulite appearance is even more impressive.



## INDICATIONS FOR USE

- Medical:
- primary lymphatic edema
  - secondary lymphatic edema
  - edema after injuries
  - chronic vein insufficiency
  - varices, artery post-operation conditions
  - shank lesions
  - prophylaxis of vein thrombosis
- Cosmetic:
- besity
  - heavy feet

## CONTRAINDICATIONS

- edema due to heart failure
- liver edema
- liver insufficiency
- thrombosis
- thrombophlebitis
- fever
- TBC
- cancer
- micro and macro angiopathies in diabetes,
- mb. Burger, mb. Rayneud

## Technical data:

Model:	<b>GREEN PRESS 12</b>
Code:	<b>1800105</b>
Voltage:	230V, 50/60 Hz
Power:	80 VA
Fuse:	2 x 400 mA
Weight:	8,2 kg
Weight with cuffs:	16,7 kg
Dimensions:	500x410x220 mm
Security class.	I
Type of applicators:	BF
Classification MDD 93/42/EEC:	Ila
No. of exits:	2 x 12
Adjustable pressure:	0 to 22 kPa
Adjustable massage parameters:	
Pressure in cuff cells:	0 to 22 kPa
Pressure duration:	0 to 90 sec

Cell-activation interval:	0 to 90 sec
Cell-activation mode:	linear – consequently one after another circular – several in waves point – one by one
Direction of massage:	towards heart or away from heart
No. of standard programs:	7
No. of free programs:	14
Basic set:	cuff for leg (Code: <b>1600102</b> ) 2 pcs cuff for foot (Code: <b>1600104</b> ) 2 pcs cuff for belt (Code: <b>1600105</b> ) 1 pc cuff for hips (Code: <b>1600106</b> ) 1 pc cuff for arm (Code: <b>1600103</b> ) 1 pc mains cable (230V Europlug) Operators Manual
Option:	cosmetic cuff (Code: <b>1600114</b> ) extension belt 10 cm (Code: <b>1600119</b> )

## DISTRIBUTOR:

## PRODUCER:

Iskra Medical d.o.o., Otoče 5a, 4244 Podnart, Slovenia  
**P:** +386 4 53 25 760, +386 4 53 25 762, **F:** +386 4 53 25 718  
**M:** +386 41 622 918, **E:** info@iskramedical.eu, **W:** www.iskramedical.eu